


# MeriScreen™

MERISCREEN Malaria Pf/Pan Ag

Evaluated  
& Pre Qualified  
by WHO

Meril  
Diagnostics

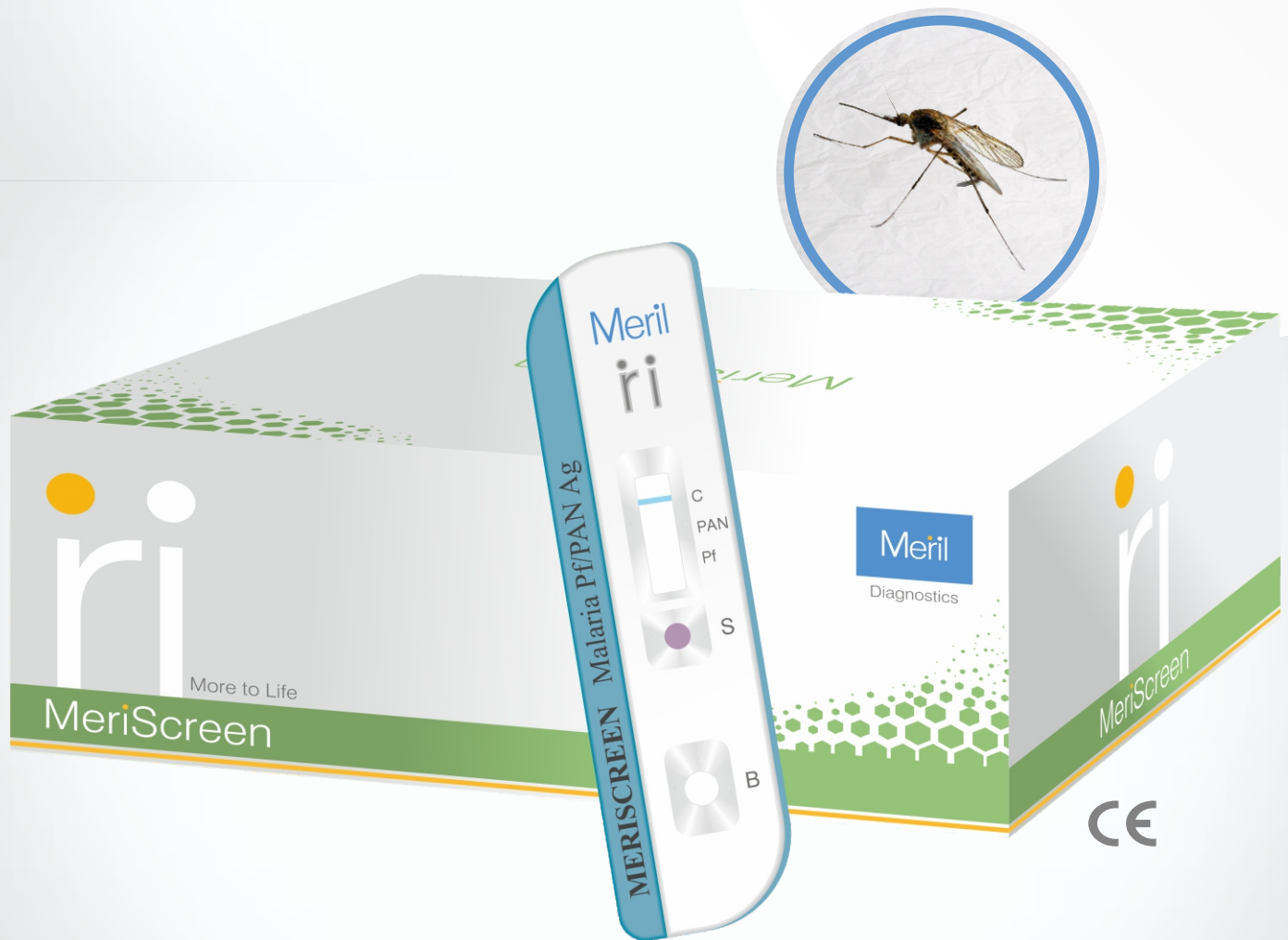

## SPECIFICATION


- Rapid test (ICT) for differential diagnosis of Malaria
- Based on HRP II & pLDH enzymes
- Shelf life of 24 months
- Specimen transfer device
- Results after 20 minutes but no later than 30 minutes
- Limit of Detection (LOD) - *P. falciparum*  $\geq 50$  parasites/ $\mu$ l and for *P. vivax*  $\geq 200$  parasites/ $\mu$ l

# MERISCREEN Malaria Pf/Pan Ag




Diagnostics

## TEST PROTOCOL

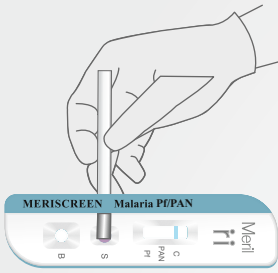

Add 5  $\mu$ l of whole blood using Specimen transfer device

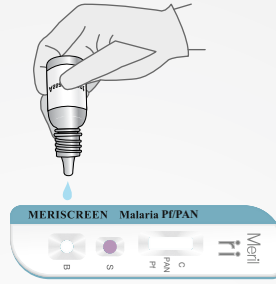

Dispense 4 Drops of Assay buffer



Results after 20 minutes but no later than 30 minutes

## INTERPRETATION

<p>Positive for <i>P. falciparum</i> or a mixed infection with <i>P. vivax</i>, <i>P. ovale</i> and/or <i>P. malariae</i></p>	<p>Positive for <i>P. falciparum</i></p>	<p>Positive for Pan (<i>P. falciparum</i>, <i>P. vivax</i>, <i>P. ovale</i> or <i>P. malariae</i>)</p>	<p>Negative</p>	<p>Invalid. Take a new cassette packaging and repeat the test</p>	<p>Invalid. Take a new cassette packaging and repeat the test</p>
---	--	--	-----------------	---	---

No Pink- Purple Line At 'C' Region Even After Addition of Sample & Assay Buffer Indicates Invalid Test

## ORDERING INFORMATION

PRODUCT CODE	DESCRIPTION	Pack Size
MHLRPD-02	MERISCREEN Malaria Pf/Pan Ag	30 Tests
MHLRPD-03	MERISCREEN Malaria Pf/Pan Ag	10 Tests
MHLRPD-04	MERISCREEN Malaria Pf/Pan Ag	25 Tests
MHLRPD-05	MERISCREEN Malaria Pf/Pan Ag	50 Tests

An ISO 13485 & GMP Certified Company



Meril Diagnostics Pvt. Ltd.

D1- D3. Second Floor. Meril Park. Survey No. 135/2/B & 174/2, Muktanand Marg, Chala, Vapi - 396191, India

Tel: +91 260 2408000 | Fax: +91 260 2408025 | Email: askinfo@merillife.com | mahendra.s@merillife.com