
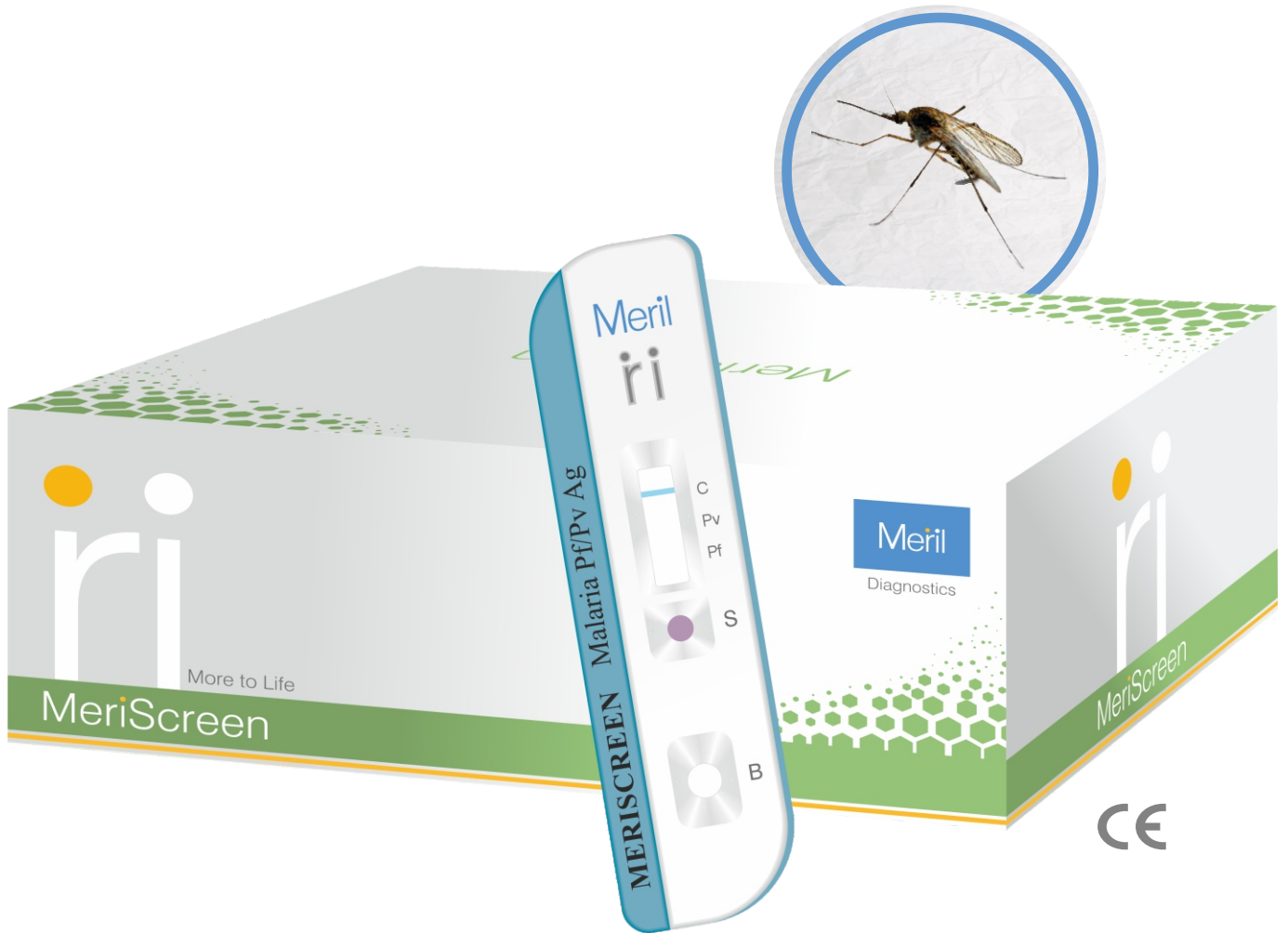


MeriScreen™


MERISCREEN Malaria Pf/Pv Ag

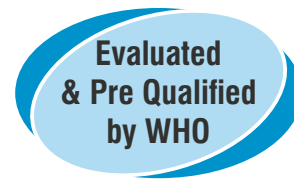


SPECIFICATION

- Rapid test (ICT) for differential diagnosis of Malaria
- Based on HRP II & pLDH enzymes
- Shelf life of 24 months
- Specimen transfer device
- Results after 20 minutes but no later than 30 minutes
- Limit of Detection (LOD) - *P. falciparum* ≥ 50 parasites/ μ l and for *P. vivax* ≥ 200 parasites/ μ l

TEST PROTOCOL

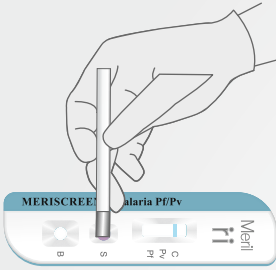

Add 5 μ l of whole blood using Specimen transfer device

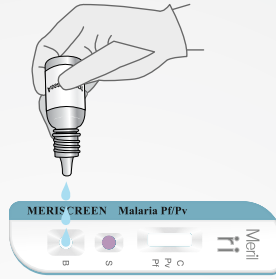


Dispense 4 Drops of Assay buffer



Results after 20 minutes but no later than 30 minutes

INTERPRETATION

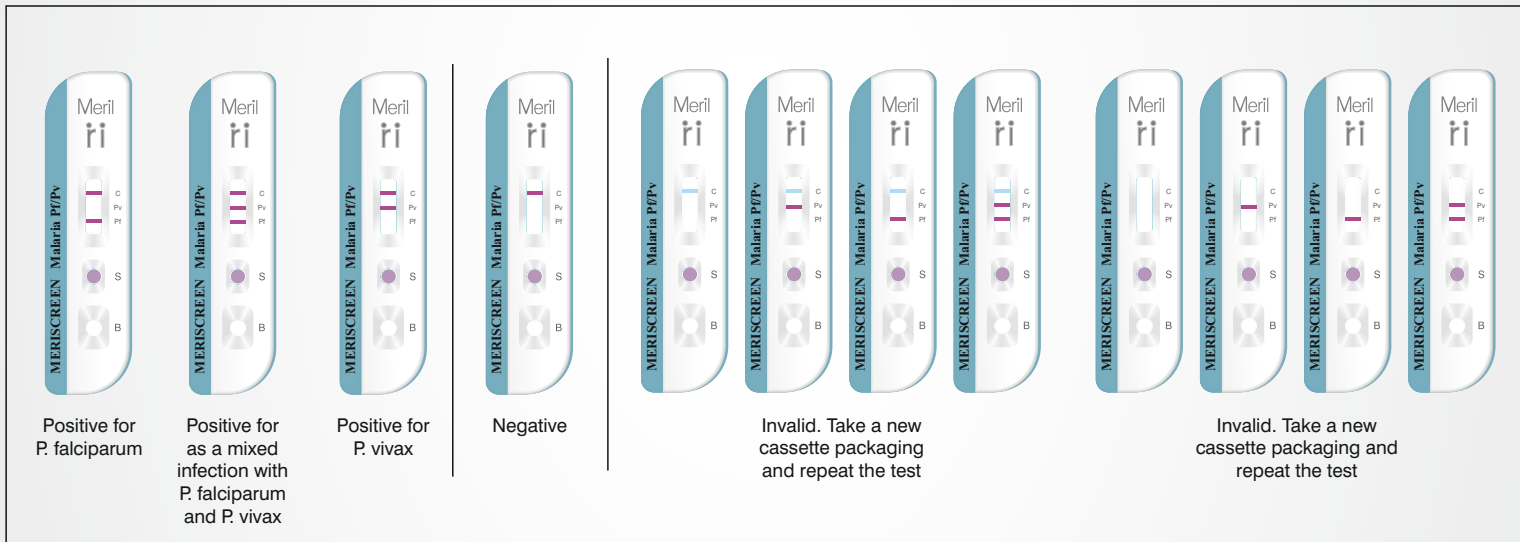

Positive for *P. falciparum*

Positive for as a mixed infection with *P. falciparum* and *P. vivax*

Positive for *P. vivax*

Negative

Invalid. Take a new cassette packaging and repeat the test

Invalid. Take a new cassette packaging and repeat the test

No Pink- Purple Line At 'C' Region Even After Addition of Sample & Assay Buffer Indicates Invalid Test

ORDERING INFORMATION

PRODUCT CODE	DESCRIPTION	Pack Size
MFLRPD-02	MERISCREEN Malaria Pf/Pv Ag	30 Tests
MFLRPD-03	MERISCREEN Malaria Pf/Pv Ag	25 Tests
MELRPD-04	MERISCREEN Malaria Pf/Pv Ag	50 Tests
MFLRPD-05	MERISCREEN Malaria Pf/Pv Ag	10 Tests

An ISO 13485 & GMP Certified Company



Meril Diagnostics Pvt. Ltd.

D1- D3. Second Floor. Meril Park. Survey No. 135/2/B & 174/2, Muktanand Marg, Chala, Vapi - 396191, India

Tel: +91 260 2408000 | Fax: +91 260 2408025 | Email: askinfo@merillife.com | mahendra.s@merillife.com